






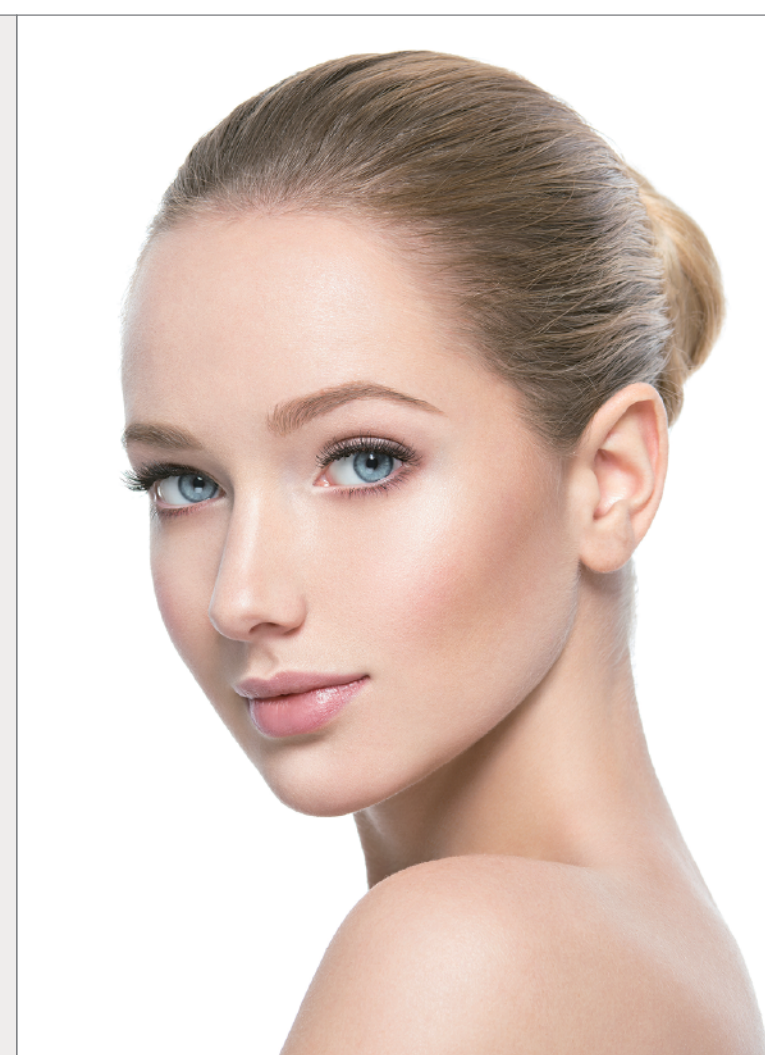
Tip Types

	Type	Shape	Area
Fractional Mode	21pin		Eye rim, nose
	77pin		Forehead, mouth, neck
	117pin		Cheek, neck
Thermal Mode	Small Tip		Eye rim, nose, mouth
	Normal Tip		Forehead, cheek, nose chin, neck

Recommended for

- Those who want to soften the skin texture
- Those who are concerned about problem skin
- Those who want skin care without interrupting daily life

TEN HI




TEN HI

Dual Mode Heat System

This product is not a medical device.
It is a reference material to help understand the product.



Fractional mode: Delivers fractionalized thermal energy
 Thermal mode: Transfer of wide heat energy to the skin

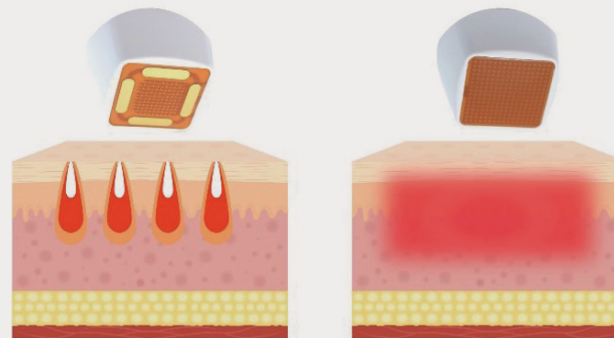


Fractional Mode

Thermal Mode

What is Ten Hi?

Ten Hi is a device that delivers thermal energy to the dermis of the skin through various types of tips. It is recommended to select a tip and mode suitable for the area to be used and operate in an optimal way.

● TEN HI ● OTHERS

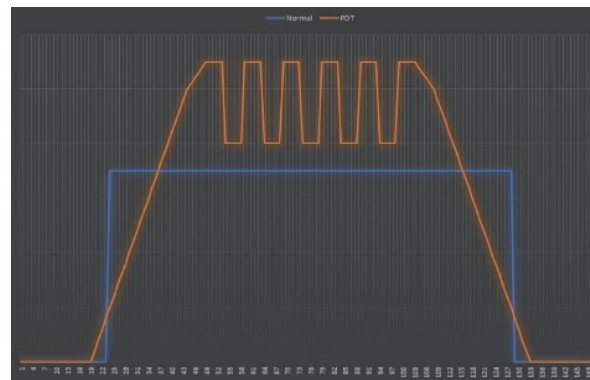

Long Pulse Mode?

Technology that divides one pulse into multiple pulses to deliver deep heat and further reduce pain.

01 When can I see the results?

Depending on the individual, you can expect immediate results immediately after use, and you will see better results after 2 to 4 weeks.

02 How often can I receive it?

We recommend 5 treatments at 2 week intervals.

03 How is the pain level?

Depends on the individual, but generally it is a tingling degree of pain. If you are sensitive to pain, lower the energy level.

04 What are the precautions?

Skin may become red, it disappears after 2 to 3 days. In the case of micro scabs, most of them naturally fall off within 3 to 7 days. It is recommended that the scab falls off on its own without removing it. You should always apply sunscreen. It is also recommended to refrain from sauna, drinking, and smoking for 3 to 5 days.